Triostar Zahnfleischmaske

Gebrauchsanleitung/ *Triostar gingival mask* *Instruction manual*

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A-Silicone for Gingival Reproduction

For the manufacture of fixed partial dentures the morphology of the gingiva is an essential reference to prevent functional and aesthetic problems. The **Triostar gingival mask** is a gingival mask based on addition curing vinyl silicones delivered in AUTOMIX-cartridges. It can be extruded directly and without any bubbles into the mould. Therefore a fast and simple manufacture of the gingival mask is possible.

The **Triostar gingival mask** supplies the essential data to the dental technicians for the determination of the cervical contours and the extreme limits for preparation of the fixed dentures. Because of its great transversal strength **Triostar gingival mask** can be easily removed from the plaster model without tearing. During the manufacture of the fixed denture, it is possible with **Triostar gingival mask** to locate and prevent the pressure onto the gingiva, which comes from the construction of the fixed denture. Therefore, it can prevent possible inflammations of the parodontium.

Gun and cartridge

Load the application gun with the AUTOMIX CARTRIDGE. Remove and dispose the closure-cap. For exact flow control extrude slightly material until uniform flow from both orifices is achieved. Install the mixing cannula onto the cartridge and extrude needed material.

Note:

Store used cartridge with fixed used mixing cannula. Do not re-use the cartridge cap. Closing the cartridge by turning on the cap between use could cause cross contamination of components and thus premature setting.

Preparation of the removable gingival masks

- The plaster model has to be prepared as usual.
- With a suitable addition curing or condensation curing impression material with a high Shore-A hardness make an impression of the relevant part of the jaw. After completely setting of the silicone, the silicone impression has to be removed.
- The trimmed stumps of model are to set free under consideration of the extreme limits for preparation.
- The silicone impression, prepared according **b)** serves as mould in conjunction with the trimmed plaster model for the preparation of the gingival mask. Within removing of the gingival contours it originates a space between the silicone impression and the trimmed model. This space has to be filled with **Triostar gingival mask** for preparation of the gingival mask. For preparation of the mould it has to be drilled two small holes side by side through the silicone part of the mould. One hole is a channel for filling of the **Triostar gingival mask**, the other one serves as a channel for escaping of the air in the mould. For an accurate and easy separation between the gingival mask and the silicone part of the mould, the silicone mould should be wetted inside with a suitable separator liquid (e.g. separator for **Triostar gingival mask**) if a condensation-curing silicone was used. If addition-curing silicone was used for preparation of the silicone mould, it is necessary to wet the silicone mould inside with the separator. With help of the mixing cannula extrude so much material of **Triostar gingival mask** until the first material comes out through the other hole. The working time (23 °C, 74 °F) of **Triostar gingival mask** is 1:30 minutes; the setting time inclusive working time (23 °C, 74 °F) is about 7 - 8 minutes. After removing of the silicone impression the gingival mask remains on the trimmed model, which references the form of the gingival borderline. Based on the prepared mask on the model the prosthetic work can be start.

Storage

Do not store above 25 °C (78 °F). Do not use after expiry date.

Additional note:

Do not use Latex gloves, they may interfere with the setting reaction of the impression material.

Warranty

Mälzer Dental GmbH & Co. KG warrants this product will be free from defects in material and manufacture. Mälzer Dental GmbH & Co. KG makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Mälzer Dental GmbH & Co. KG sole obligation shall be repair or replacement of the Mälzer Dental GmbH & Co. KG product.

Limitation of Liability

Except where prohibited by law, Mälzer Dental GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!