

# Installation instruction

- **ADESSOSPLIT**® plate set Candulor®
- Splitex® plate set Candulor®





Candulor® articulator 116 mm with extension set (art.no. 10026).



Lift Candulor® plate using a hot air gun (less than 80°C). Use a plaster knife to disengage the plate carefully.



Loosen two screws and lift the upper part of the articulator.  
Attention: Do not use the screws again.



Replace the existing silver Candulor® anterior guide pin and fix the new red pin.



Place the red block of the extension set on the articulator – text side up.



Fix the upper part of the articulator with the new, longer screws.



Warm up the Candulor® plate and press is manually to its previous position.



Articulator is ready for its calibration.



Fill metal glue in the recesses of the base plate marked with „glue dot“ and open the articulator, turning it upside-down.



The base plate will be fixed in the upper part of the Candulor® plate magnetically, supported by the metal glue.



Invert the centring key together with the calibration base plate in the articulator and apply metal glue to the center of the plate.



Close the articulator (still upside down).



Fix the articulator in place using a rubber band and let the adhesive cure for approx. 90 min.



Ready-to-use Candulor® articulator with 126 mm Artex® dimensions.

## Legal information

All information in this documentation has been prepared with great care and is subject to changes and mistakes. We assume no legal responsibility on consequences of incorrect information or improper use.

Candulor® is a registered trademark of CANDULOR AG, 8152 Glattpark, CH  
Splitex® und Artex® are registered trademarks of Amann Girsbach GmbH, 75177 Pforzheim, DE  
Adesso Split® is a registered trademark of K. Baumann, 75210 Keltern, DE