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Instruction manual

Mode d'emploi

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Accessories-Articles for reorder

<p>1 Removable plate with magnetic closure for model CORSOART® BKR, red Removable plate with magnetic closure for model CORSOART® BKR, blue Removable plate with magnetic closure for model CORSOART® BKR, gold</p> <p>2 Mounting screw for anterior tooth guide pin-Plate & pin holder</p> <p>3 Anterior tooth guide pin holder for CORSOART® A-Line, red Anterior tooth guide pin holder for CORSOART® AC-Line, blue Anterior tooth guide pin holder for CORSOART® S-Line, gold</p> <p>4 Anterior tooth guide pin screw</p> <p>5 Mounting screw-kit for CORSOART® A-Line Mounting screw-kit for CORSOART® AC-Line Mounting screw-kit for CORSOART® S-Line</p>	<p>Art-No.: 12167-M Art-No.: 12267-M Art-No.: 12367-M</p> <p>Art-No.: 12406</p> <p>Art-No.: 12142 Art-No.: 12242 Art-No.: 12322</p> <p>Art-No.: 12407</p> <p>Art-No.: 12402 Art-No.: 12403 Art-No.: 12404</p>	<p>6 Incisalneedle for CORSOART® articulators Art-No.: 12408</p> <p>7 Anterior tooth guide pin for CORSOART® A-Line Anterior tooth guide pin for CORSOART® AC-Line Anterior tooth guide pin for CORSOART® S-Line Art-No.: 12321</p> <p>8 Plate plain 0° for CORSOART® A-Line & Artex®, red Plate plain 0° for CORSOART® AC-Line & Artex® Carbon, blue Plate plain 0° for CORSOART® S-Line & SAM®, gold</p> <p>9 Support pin, screwable for CORSOART® A-, AC- and S-Line Art-No.: 12401</p> <p>10 Tool key for CORSOART® BKR articulators Art-No.: 12421</p>
--	---	--

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
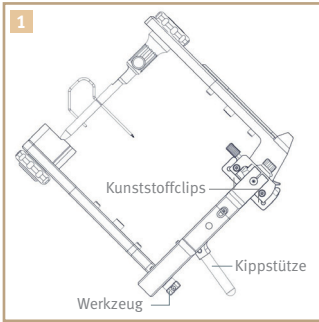
Instruction manual CORSOART® articulators


45° tilt support/tool

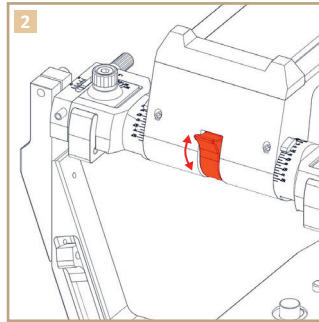The tilt supports are located at the back of the articulator. This lets the articulator be tilted back to less than 45°. The tool for the following functions is located in the base of the articulator frame.


Opening/closing the centric

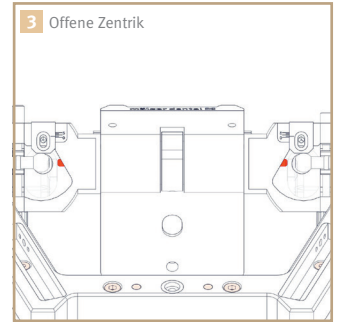To open the centric, move the swivel arm up as far as it will go – this releases the centric. To close the centric, move the swivel arm in the opposite direction – this will again fix the condyles in place in a central position.

Opening the centric

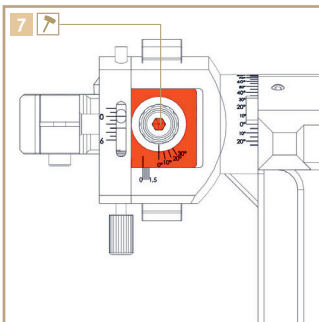
The condyles are no longer fixed and the centric is released. The top half of the articulator can be moved freely.

Immediate side shift (ISS)


Loosen the locking screw and set the ISS to the desired position.

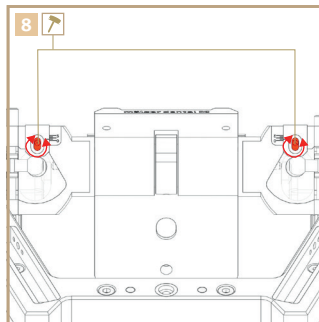Can be set from 0 mm to 1.5 mm per side

Setting protrusion/retrusion

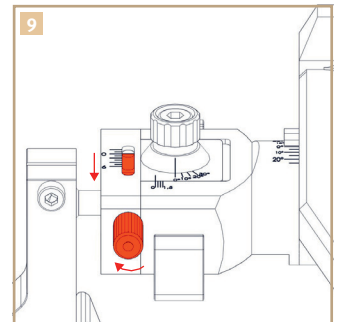To change either of these settings, first loosen the locking screw on the block underneath the top half of the articulator.

Setting protrusion

Protrusion can be set after loosening the locking screw.

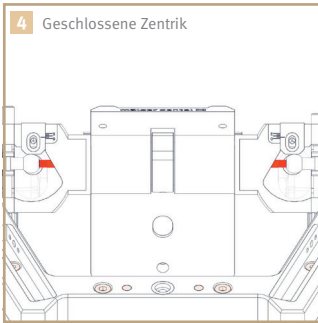
Can be set from 0 mm to 6 mm

General information & care instructions

Precision instruments such as articulators must comply with the highest requirements. Therefore, the following instructions for use of these devices should be strictly observed.


- Regular inspection of the axis-ball-system and the condylar path for contamination. This secures the accurate functionality of the articulator and maintains the smooth running of the ball on the axis.
- Contamination that took place here, can be removed by evaporation. Subsequently blow dry remaining moisture.
- Under no circumstances may the articulator be put in an extraction unit, pressure pot or ultrasonic bath.

- Use only cleaning agents that are suitable for the cleaning of light metals and apply according to the manufacturer's instructions. Cleaning agents, which have a pH value of below 5 and above 8, may not be used, in order to prevent damage to the anodised surfaces.
- Please, do not use a plaster dissolution, in order to prevent damage to the anodised surfaces.
- For cleaning and care, please, use only the special care spray (Art.-No. 12015).

Closing the centric

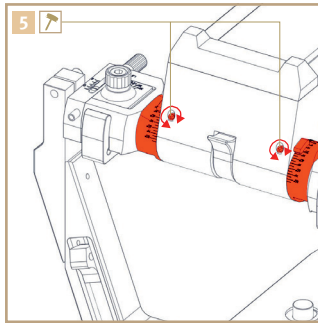
The condyles are again fixed in place, the centric is visibly closed.

Condylar path inclination

Loosen the two HCl set screws with the centric closed and set the angle for the condylar path inclination on the condylar housing. Re-tighten the HCl set screws after setting the angle.

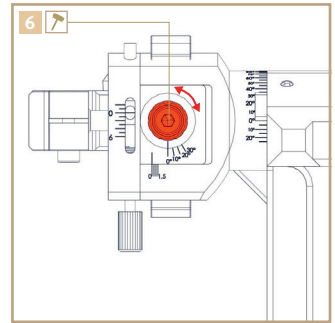
Angle can be set from -25° to $+80^{\circ}$

Setting the Bennett angle

Loosen the locking screw and set the Bennett angle to the desired position.

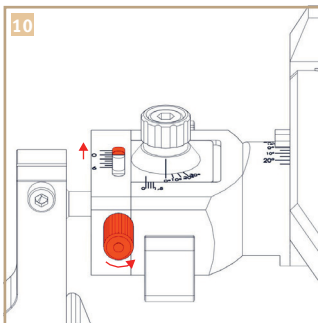
Angle can be set from 0° to $+30^{\circ}$

Setting retrusion

Retrusion can be set after loosening the locking screw.

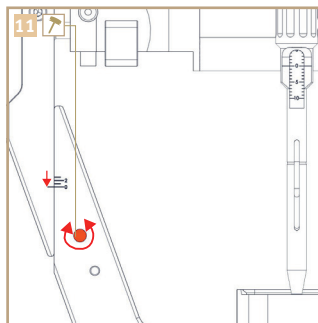
Can be set from 0 mm to 2 mm

Setting distraction

Distraction can be set after loosening the locking screw on the side of the articulator.

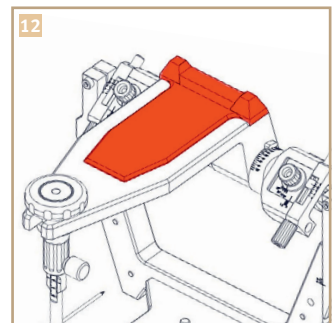Can be set from 0 mm to 3 mm

Removable plates

The removable plate is secured with a magnetic closure and can be removed simply by lifting up and out.

Legal information

All information in this documentation has been prepared with great care and is subject to changes and mistakes. We assume no legal responsibility on consequences of incorrect information or improper use.

Tool required

The special tool is located at the back of the bottom half of the articulator.

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