Pinbohrgerät M23 compact

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Pin drilling unit M23 compact

 Instruction manual

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Warning: Check that all components are complete and undamaged before use!

Technical specifications

- Dimensions D/W/H: 270 x 190 x 390 mm
- Weight: 6.5 kg
- Connected loads: 55W 230V 50-60 Hz
- Rotary speed: 3,600 RPM
- Laser class: 3 A < 5 mW

Package contents

- **DIVARIO**® pin drilling unit M23 compact (art. no. 14523)
- **DIVARIO**® carbide burr, 1 pc (art. no. 10171)
- **DIVARIO**® Duo plate support (art. no. 14083)
- **DIVARIO**® Impression carrier (art. no. 14090)
- Assembly
- Power plug

Warning and safety instructions

- Never change the burr when the machine is switched on
- Keep fingers and hands away from the area of the laser dot when the machine is switched on – danger of injury!
- The pin drilling unit M23 compact is not designed for use as follows:
 - In explosion hazard areas
 - For medical applications
 - For working with damp or wet materials
- The Mälzer Dental product warranty will be voided by any handling of the pin drilling unit M23 compact other than that specified in the user manual
- Observe the regulations of the relevant employer's liability insurance association when using the M23 compact pin drilling unit:
 - Always wear protective goggles
 - Ensure adequate workplace lighting
 - Use an extraction unit
- **Warning:** All repairs or other work on the machine must be carried out solely by Mälzer Dental or qualified personnel authorised by Mälzer Dental!

General information & care instructions

- Before commissioning, check that the mains power specifications match those on the rating plate.
- Clean the pin drilling unit from time to time with a soft cloth. Do not use compressed air or detergents.
- If the machine develops a fault, always return the unit directly to Mälzer Dental or an authorised dealer.
- High-quality technical machinery of this kind requires an appropriate repair service and specially trained personnel. Mälzer Dental guarantees that repairs will be performed properly with original replacement parts and offers a six-month guarantee on all repairs!

Legal information

The information presented here is correct to the best of our knowledge. Errors and omissions excepted. We accept no legal responsibility for the consequences of inaccurate statements or improper usage.

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Commissioning

Attention – do not lift the pin drilling unit by grasping the laser arm but solely under the magnet plate!

Electrical connection, incl. power plug (220 V/320 W) and main switch used to switch the pin drilling unit off and on.

Release button for starting the drilling procedure.

Setting the drilling depth

Note: the optimum drilling depth is factory-set!
The drilling depth is regulated by using an Allen key to rotate the set screw located under the burr (centre of the axis) to the chosen position.

Clockwise rotation: Screw is lowered = drilled hole is narrower

Anticlockwise rotation: Screw is raised = drilled hole is wider

Changing the burr

Switch off and unplug the machine. Using the Allen key provided, disengage the grub screw to the side of the brass spindle and pull the burr upwards to remove.

Now insert the new burr from above and retighten the grub screw. To set the drilling depth, proceed as shown in Figure 5.

Switch the machine back on and carry out a test drill.


Placing and blocking-out the impression + drill procedure

Trim the impression, blocking-out missing areas with wax or silicone.


Lie the **DIVARIO® Positionierung plate** parallel to the impression.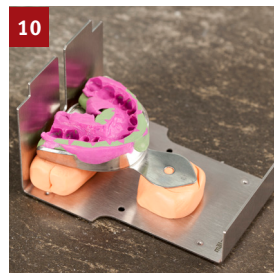

Fix the impression with **DIVARIO® putty** onto the **DIVARIO® impression carrier L**.


To orient the impression, use the small or large **DIVARIO® positioning plate**.

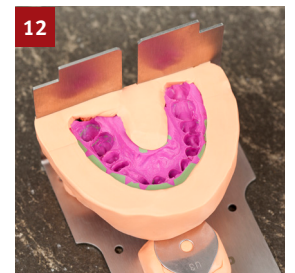Block-out missing areas with **DIVARIO® putty**, so as to avoid leaks of plaster mush when pouring out.


Clamp the **DIVARIO® base plate** into the **DIVARIO® Duo plate support** on the **DIVARIO® M23 compact pin drilling unit**.

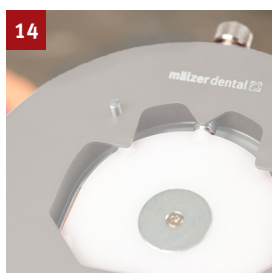
Clamped **DIVARIO® base plate**.

Place the **DIVARIO® impression carrier L** onto the **DIVARIO® Duo plate support**.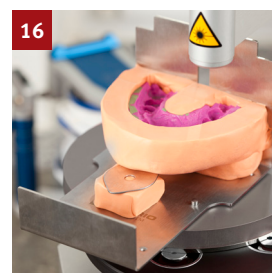

Place the **DIVARIO® impression carrier L** with the **DIVARIO® Duo plate support** onto the magnetic plate on the **DIVARIO® M23 compact**.

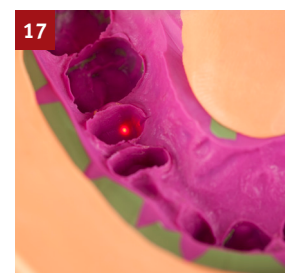Use the laser pointer to set/target the required drilling position.

Press the button to start drilling – the **DIVARIO® Duo plate support** is held in place electromagnetically.

DIVARIO® base plate with drilling complete.

Prepare the **DIVARIO® base plate** with **DIVARIO® pins**.

Once pinned, check the **DIVARIO® base plate** by placing it in the impression.

Pouring out and finishing

Fill the impression with plaster mush. To improve flow properties, increase shaker vibration strength as required.

To avoid trapping air bubbles when immersing the **DIVARIO® base plate**, fill the impression with more plaster mush than necessary.

Place the **DIVARIO® base plate** upside down in the carrier plate guide and lower gently.

Allow a little time to set and then carefully remove excess plaster around the **DIVARIO® base plate**.

After approx. 25 minutes, carefully remove the impression together with the **DIVARIO® base plate**.


Demoulding, trimming and sawing

Tap carefully to free the dental model from the **DIVARIO® base plate**.

An alternative to a model trimmer, work the dental model freehand with a **DIVARIO® finishing strip**.

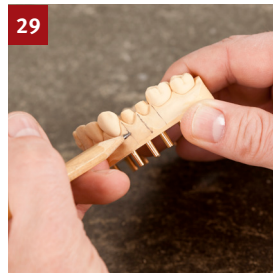Mark sections beforehand for better orientation.


Section the dental model with a **DIVARIO® Diacut dental cutting disc**.

Work the individual segments with a **DIVARIO® finishing strip** and clean to remove plaster dust.


Completed **DIVARIO®** sectioned model, seated exactly on the **DIVARIO® base plate**.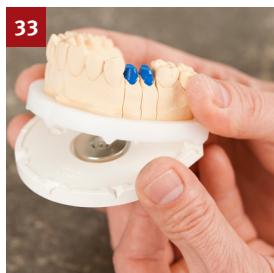

Position the **DIVARIO® split-cast plate** before mounting so as to guarantee a perfect fit.

Finished, mounted work in the **CORSOART® articulator**.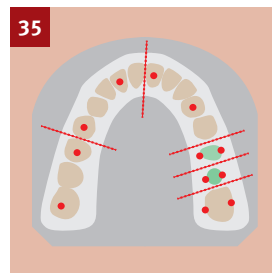

For large segments, mark additional relief cuts (see sketch). Pins are represented by red dots in the sketch.

Schlesierweg 27 · D-31515 Wunstorf (Germany)
Phone: +49 (0) 50 33 / 963099-0 · Fax: +49 (0) 50 33 / 963099-99
E-Mail: info@maelzer-dental.de · Internet: www.maelzer-dental.de

maelzer dental 