Separator Zahnfleischmaske Gebrauchsanleitung/ *Separator for Triostar gingival mask* *Instruction manual*

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Instruction manual

Separator for Triostar gingival mask

The **separator** for Triostar gingival mask is a separating agent for A-Silicones. It prevents adhesion of two silicones and is well recommended.

The use of the **separator** for Triostar gingival mask ensures a good separation of the silicones after setting.

The **separator** is a spray, consisting of paraffines in aliphatic hydrocarbons. The separator is only for extraoral applications.

Application

The **separator** for Triostar gingival mask should only be used under an exhaust hood or with a suitable suction device!

For an accurate and easy separation between the gingival mask and the silicone part of the mould, the silicone mould should be wetted inside with the separator if a condensation-curing silicone was used.

If an addition-curing silicone was used for preparation of the silicone mould, it is necessary to wet the silicone mould inside with the **separator**.

Spray the **separator** with a distance of approx. 15 cm on all relevant areas of the impression and allow to take effect for about 20 seconds.

Then apply the Triostar gingival mask immediately - as described in the corresponding instructions - into the impression. After setting, the material and the impression can be separated easily from each other.

Storage

Do not store above 25°C (77°F). Do not use after expiry date.

Warnings

- Container is pressurized. Do not open with force even after use.
- The separator contains n-hexane, propane and butane. Highly flammable, harmful, danger of serious damage to health by prolonged exposure through inhalation. Dangerous for the environment.
- Do not breathe substance.
- Do not spray into eyes.
- Protect from sunlight and do not expose to temperatures exceeding 50°C (122°F).
- Use only for intended purpose.
- Keep away from sources of ignition – no smoking.
- In case of contact with eyes rinse with plenty of water, In case of contact with skin wash with water and rinse thoroughly.

Additional note:

Do not use Latex gloves, they may interfere with the setting reaction of the impression material.

Warranty

Mälzer Dental GmbH & Co. KG warrants this product will be free from defects in material and manufacture. Mälzer Dental GmbH & Co. KG makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Mälzer Dental GmbH & Co. KG sole obligation shall be repair or replacement of the Mälzer Dental GmbH & Co. KG product.

Limitation of Liability

Except where prohibited by law, Mälzer Dental GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!